

Lumex HybridLX Rollator Transport Chair

Meeting the Needs of the End-User

The population of Baby Boomers (people who were born between the years of 1946 and 1964) has increased dramatically over the past couple of years. This year that number is higher than ever. An even more interesting fact is that these Baby Boomers of today are more active and are seeking more and more years of an independent lifestyle. So both they and their family members are not only searching for the products they need that are basic, but they are moving towards products that keep them active, independent, less “old” or institutional feeling and out of the nursing home. What they are looking for is something that is not only functional but aesthetically appealing that they can use in the comfort of their home and when they are out and about. And that is where a product like the **Lumex® HybridLX Rollator Transport Chair** comes in.



.....

The history of patient aids has evolved from a cane, to a walker, to a rolling walker (aka rollator) to now the hybrid product of a rollator/transport chair. What does the Lumex HybridLX offer to the end-user?

- ❖ **Design Concept**– the Hybrid LX gives an ideal solution to those looking for a product that meets their needs of providing support when walking and a seat but also provides the additional benefit of transforming into a transport chair that can be used to move the end-user when they get tired. It is very common for older people to tire easily so why not give them a product that they use when they want to be independent and when they need that extra resting period. Older people will wait as long as they can before having to use a transport chair or wheelchair because it makes them feel old and limited to what they can do. The Hybrid LX is designed to meet the needs of the end-user but in an innovative design that they can be proud to use.
- ❖ **Aesthetically Appeal** – The look of the Hybrid LX is geared towards that of a more modern, upscale, durable and high-end feel. From the colors, to the functionality to the shape and contours of the frame, it was designed to stand out and not look like the traditional rollator.

Anyone can make this!



This is different!



- ❖ **Durable and Comfortable Handgrips** – provides users with a durable and secure grip that lasts longer than traditional foam grips
- ❖ **Easy-to-Use Handbrakes** – brakes are designed for multi-purpose use. Handbrakes can be used during rollator use mode and be pulled up for quick stops. When user is getting ready to sit, handbrakes should be locked by pushing down to keep rollator stable when sitting.



- ❖ **Flexible Backrest** – padded backrest provides comfort and flexibility without having to have a rigid backbar. Also backrest can be easily adjustment when switching from rollator to transport chair mode without having to remove or re-adjust any components.



- ❖ **Comfortable Seat** – 18” wide seat gives the user more seat area. Seat is designed to allow for easy folding of the Hybrid LX and has a center seat strap that can be pulled to assist in folding. Product has been tested and has a 300 lb weight capacity.



- ❖ **Detachable Footrests** – the footrests were designed similar to current footrests on transport chairs. They can be easily attached and are detachable should the end-user not want to use the product as a transport chair. **Key point:** The footrests can also be folded back and locked onto the sides of the product allowing the end-user to keep them on if they intend on using the product as multi-functional more frequently. They can be adjusted in length depending on patient height and does not require tools to adjust and have a heel strap to support the patient’s feet if needed.



- ❖ **Height Adjustability** – uses simple push button to adjust and secure to desired user height. Broad range of users from 5’ 4” – 6’2” can use this product.



- ❖ **Storage Pouch** – Convenient storage pouch provides user with area to place personal items.



- ❖ **Wheels** – 8” wheels made of PVC are durable, non-marking. Wheels are designed for indoor and outdoor (rugged terrain) use for easy riding. Front wheels are removable using the bright red push button and easily snap into place. Rear wheels can be easily folded for storage. Simple screw knob that requires little dexterity secures wheels into place. *Important Note: Drive’s unit has removable rear wheels which are more difficult for user to assemble. The HybridLX only requires unfolding and inserting of the screw knobs.*



- ❖ **Color Offerings** – Offered in both Titanium and Majestic Blue that are attractive and eye appealing versus traditional industry colors.
- ❖ **Convenient Storage** – A convenient carry pouch is available for purchase should user want to neatly store the HybridLX.



For the Youthful and Active Individual on the Go



Visit the website at www.grahamfield.com for more information.